

Installation instructions for Kröber service software

Hardware requirements:

- PC with Windows 2000 / XP / Vista / 7
- USB interface
- Printer

Step-by-Step instruction

1. Download and decompress setup file KroeberO2_x_x_x_x.zip.
(x_x_x_x represents the actual version)
2. Connect the KroeberO2 to the USB interface of your PC and on the concentrator.
3. Windows detects the new hardware and automatically installs the necessary drivers.
If you are asked to locate the drivers, point to the folder CDM20812 in the destination folder.
4. Install the KroeberO2 service software by starting setup.exe in the destination folder.
5. Start KroeberO2 after successful installation in the start menu.
6. Enter the registration code.
7. The installation processed is finished. The service software establishes a connection to the KroeberO2.